



# EMIL

AK MedTec



AK MedTec

# EVA

## Quick Facts




### SYSTEM CHECK

On/Off press briefly

- green continuous light > battery full
- green light flashes > battery between 75 and 35%
- red continuous light > battery below 35% (no stress, running time easily 2 x 45 minutes)
- red light flashes > system defective

### START

Press On/Off for 2 seconds

- System starts at the highest heating level (40°C) 
- press again and regulate to 36°C 
- press again and regulate to 32°C 

### RUNNING SYSTEM

- Temperature is reached after 2 - 3 minutes. Attention: Only with patient (congestion heat)
- Temperature monitored by sensors
- Automatically switches off after 45 minutes

### BATTERY RUNNING TIME/CHARGING TIME

- at -10°C approx. 3 - 3.5 hours running time (for an average 75 kg patient) in the highest heating setting, correspondingly longer for heavier patients
- empty battery > charging time is 3 hours with 220 V

### CLEANING

- When the system is closed, it is water and splash proof



AK MedTec

# ELMO



AK MedTec

# EMMA



AK MedTec

# Heated Medical Devices

[Download  
Manual](#)

